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Intended use

The "Urine, rectal swab and buccal swab sampling set" from Deutsche Aidshilfe / s.a.m Health is used as a set for taking and sending a urine sample, as well as swabs from the rectum and oral mucosa, for suitable and validated laboratory tests. It can be used by healthcare professionals as well as for self-testing. For *in vitro* diagnostic purposes only.

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Warnings and precautions

- *Not suitable for persons younger than 18 years.*
- *Use the products of the sample collection set exclusively like described in the instructions.*
- *Don't use the sample collection set after exceeding the best before date (last day of the month and year written on the packaging).*
- *Keep the alcohol pads away from fire and potential sources of ignition.*
- *Don't bring the alcohol pads in contact with your eyes or mucous membranes and don't breath in the released fumes.*
- *By no means should the products of the sample collection set be used more than once.*
- *Do not apply the transport medium directly to skin or mucous membranes or take internally.*
- *Do not excessively press the swab during collection, as this may result in accidental breakage of the shaft.*
- *Self-collection of the swabs must not be performed if the sampling area has recently undergone surgery or if there is unusual bleeding or pain.*
- *Rectal self-collection must be performed without prior disinfection or lubrication, at least 48 h after last sexual intercourse, in the absence of any local treatment.*
- *Any serious incidents that occur should be reported to the procedure pack producer and the competent authority.*
- **Emergency contact information: In emergencies, you can reach the medical on-call service by dialing 116 117. In life-threatening situations, please dial the emergency number 112.**

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Test activation

- Log in to your account and click on "Activate test kit".
- Select the appropriate test kit from the list.
- Scan the barcode on the test kit with your smartphone or your computer's camera.
- Alternatively, you can also enter the number below the barcode manually.

Without activating the test kit, the samples sent in cannot be assigned to you. In this case a new test kit must be ordered at your own expense.

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Prepare all materials for sampling and carry out the sample collections one after the other as shown. The swabs are placed in the yellow urine sample tube.

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Buccal swab

Attention: Do not eat, drink (except water), smoke or brush your teeth up to 1 hour prior sample collection.

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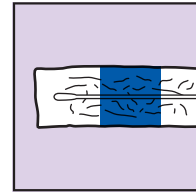
Rectal swab & Urine collection


You should not urinate for at least 1 hour before urine collection to avoid falsifying the result. The intimate area should not be cleaned before collection. The rectal self-collection should take place no earlier than 48 hours after the last sexual intercourse and no rectal irrigation should be performed 2 hours beforehand.

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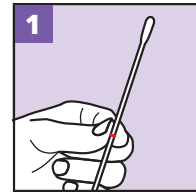
For the buccal swab, use the swab with the **blue back**, that shows collection in the oral cavity. Have the **yellow sample tube of the urine collection** set ready.

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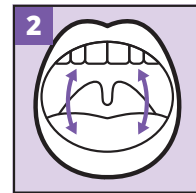
Hold the swab on the **red marking** for the sample collection. Do not touch the sample collection side of the swab or drop it anywhere.

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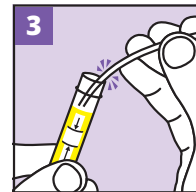
For the sample collection, place the swab in the mouth and carefully turn it a few times on the **inside of the cheek**.

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
Place the swab with the **sampling tip ahead in the urine sample tube** and bend it 180° over the edge until it breaks off at the pre-determined breaking point. If necessary, carefully turn the swab.

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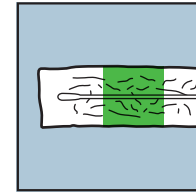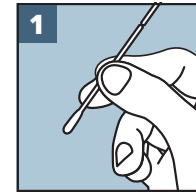
For the rectal swab, use the swab with the **green back**, that shows rectal sampling. Have the **yellow sample tube of the urine collection** set ready.

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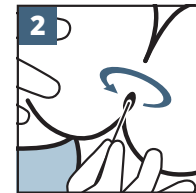
Remove the swab from the packaging and hold it by the **red marking** (see illustration). Do not touch the sample collection side of the swab or drop it anywhere.

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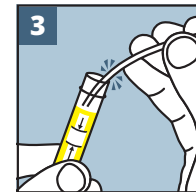To make removal easier, place one foot on a raised surface (e.g. chair). Insert the swab about **2 cm to 3 cm** deep into the anus. After **2 to 3 circular movements**, slowly pull the swab out.

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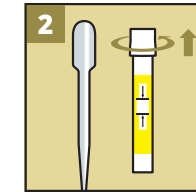
Put the swab into the urine sample tube as well. Therefore again place the swab with the **sampling tip ahead in the urine sample tube** and bend it 180° over the edge until it breaks off at the pre-determined breaking point. If necessary, carefully turn the swab.

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
Collect your **first-catch urine** in a dry, clean container (e.g. cup). To do this, urinate in the container **for a maximum of 5 seconds**.

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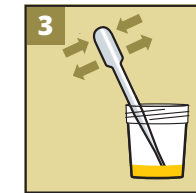
Take the **pipette** from the packaging. Prepare the urine sample tube with the **two swabs already included**. Use the sample tube with the swabs that have already been collected.

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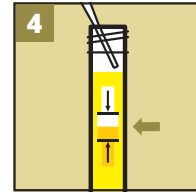
Transfer the urine into the sample tube using a pipette. The pipette is **compressed** (air escapes) and the urine is drawn up by slowly **releasing** the pressure.

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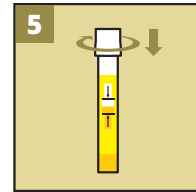
Transfer the urine into the sample tube by compressing the upper pipette part again. The tube should be filled up to the **middle of the designated area**.

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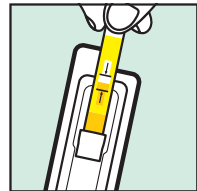Insert the test tube into the **shipping container** and seal it. Send the **shipping container** in the prepaid **return envelope**.