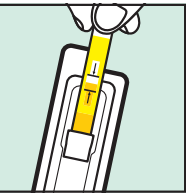
Versand Shipping

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As soon as your result is available in your account, you will receive a notification from s.a.m health. If the result is negative, you can view it directly. If the result is reactive/ positive, you will receive a notification asking you to call your checkpoint.

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If you encounter any problems with the test kit (e.g., defects, missing parts, unusual reactions), please report them immediately to: sam-vigilance@dah.aidshilfe.de – with the batch number (see packaging) and a brief description of the problem.

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Intended use

The "Urine, dried blood, rectal swab and buccal swab sampling set" from Deutsche Aidshilfe / s.a.m health is used as a set for taking and sending urine and dried blood samples, as well as swabs from the rectum and oral mucosa, for suitable and validated laboratory tests. It can be used by healthcare professionals as well as for self-testing. For in vitro diagnostic purposes only.

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Warnings and precautions

- *Not suitable for persons younger than 18 years.*
- *Use the products of the sample collection set exclusively like described in the instructions.*
- *If you suffer from a bleeding disorder or take anticoagulant medication, you are pregnant or breast feeding, please talk to your physician before using the blood collection set.*
- *Don't use the sample collection set after exceeding the best before date (last day of the month and year written on the packaging).*
- *Keep the alcohol pads away from fire and potential sources of ignition.*
- *Don't bring the alcohol pads in contact with your eyes or mucous membranes and don't breath in the released fumes.*
- *Check the cap of the safety lancet before use and don't use the lancet in case of damage.*
- *By no means should the products of the blood collection set be used more than once.*
- *Please don't touch the spots of the blood spot card where the blood should be applied.*
- *The dried blood spot card should not come into contact with any other liquids.*
- *Do not apply the transport medium directly to skin or mucous membranes or take internally.*
- *Do not excessively press the swab during collection, as this may result in accidental breakage of the shaft.*
- *Self-collection of the swabs must not be performed if the sampling area has recently undergone surgery or if there is unusual bleeding or pain.*
- *Rectal self-collection must be performed without prior disinfection or lubrication, at least 48 h after last sexual intercourse, in the absence of any local treatment.*
- *Any serious incidents that occur should be reported to the procedure pack producer and the competent authority.*
- **Emergency contact information: In emergencies, you can reach the medical on-call service by dialing 116 117. In life-threatening situations, please dial the emergency number 112.**

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Test activation

- Log in to your account and click on "Activate test kit".
- Select the appropriate test kit from the list.
- Scan the barcode on the test kit with your smartphone or your computer's camera.
- Alternatively, you can also enter the number below the barcode manually.

Without activating the test kit, the samples sent in cannot be assigned to you. In this case a new test kit must be ordered at your own expense.

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Prepare all materials for sampling and carry out the sample collections one after the other as shown. The swabs are placed in the yellow urine sample tube.

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Blood collection

Wash hands with warm water and soap, rub hands and massage the finger with gentle pressure from palm to fingertip. Let arm hang after punctation.

Buccal swab

Attention: Do not eat, drink (except water), smoke or brush your teeth up to 1 hour prior sample collection.

Rectal swab & Urine collection

You should not urinate for at least 1 hour before urine collection to avoid falsifying the result. The intimate area should not be cleaned before collection. The rectal self-collection should take place no earlier than 48 hours after the last sexual intercourse and no rectal irrigation should be performed 2 hours beforehand.

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
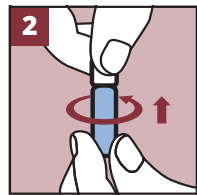
Wash hands thoroughly with water and soap and clean the middle or ringfinger with the included alcohol pad (use no other disinfectant).

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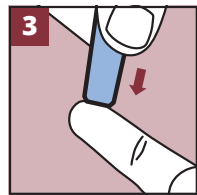
Open the included safety lancet by turning and pulling of the white cap.

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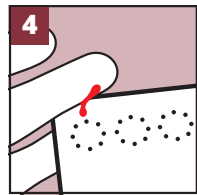
Apply the lancet at the side of the fingertip. Due to this pressure the lancet will automatically activate. Let arm hang after punctation. Each lancet should and can **only be used once**.

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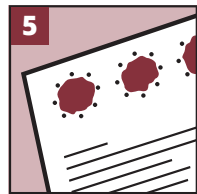
Wipe off the first drop of blood with the compress. Drip the following droplets on the preprinted circles on the dried blood spot card.

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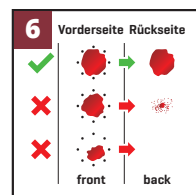
Fill all circles of the dried blood spot card. Apply the blood on the front side. The bleeding can be treated with the included compress and plaster. If necessary, use the second lancet to puncture again.

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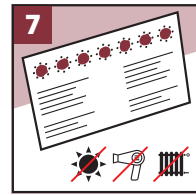The circles should be **sufficiently filled and soaked** (area $\varnothing > 6$ mm per circle). The card should look approximately the same on both sides.

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

Let the dried blood spot card dry for at least **2-4 hours** at room temperature by leaning the card against an object. Don't use a heating source for drying.

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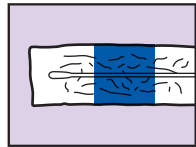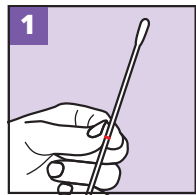
For the buccal swab, use the swab with the **blue back**, that shows collection in the oral cavity. Have the **yellow sample tube of the urine collection** set ready.

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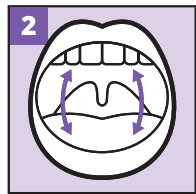
Hold the swab on the **red marking** for the sample collection. Do not touch the sample collection side of the swab or drop it anywhere.

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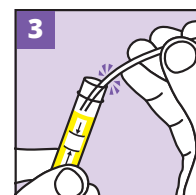
For the sample collection, place the swab in the mouth and carefully turn it a few times on the **inside of the cheek**.

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
Place the swab with the **sampling tip ahead in the urine sample tube** and bend it 180° over the edge until it breaks off at the predetermined breaking point. If necessary, carefully turn the swab.

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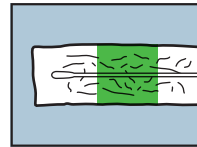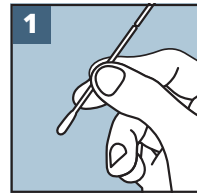
For the rectal swab, use the swab with the **green back**, that shows rectal sampling. Have the **yellow sample tube of the urine collection** set ready.

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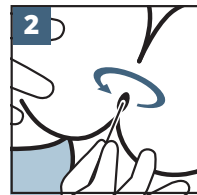
Remove the swab from the packaging and hold it by the **red marking** (see illustration). Do not touch the sample collection side of the swab or drop it anywhere.

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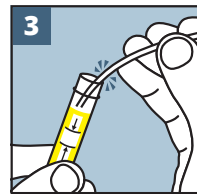To make removal easier, place one foot on a raised surface (e.g. chair). Insert the swab about **2 cm to 3 cm** deep into the anus. After **2 to 3 circular movements**, slowly pull the swab out.

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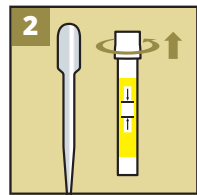
Put the swab into the urine sample tube as well. Therefore again place the swab with the **sampling tip ahead in the urine sample tube** and bend it 180° over the edge until it breaks off at the predetermined breaking point. If necessary, carefully turn the swab.

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Collect your **first-catch urine** in a dry, clean container (e.g. cup). To do this, urinate in the container **for a maximum of 5 seconds**.

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
Take the **pipette** from the packaging. Prepare the urine sample tube with the **two swabs already included**. Use the sample tube with the swabs that have already been collected.

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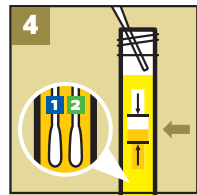
Transfer the urine into the sample tube using a pipette. The pipette is **compressed** (air escapes) and the urine is drawn up by slowly **releasing** the pressure.

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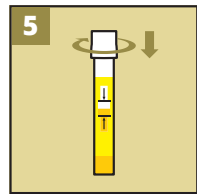Transfer the urine into the sample tube by compressing the upper pipette part again. The tube should be filled up to the **middle of the designated area**.

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